

User Instructions Economy Shower Chair Range

Maximum
User
Weight:



General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

1.0 Intended Use

This range of shower chairs is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.

2.0 Safety Precautions

- Do not exceed the maximum user weight.
- Ensure seat is correctly fitted and secure.
- Inspect product at regular intervals see section 5.2
- Do not use if feet or castors are damaged.
- Ensure all castors are locked when the seat is in the desired position and if leaving the client temporarily.

3.0 Unpacking

Check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

4.0 Assembly and Height Adjustment

- Where the seat is supplied separately, fit to the frame by first clipping it onto the rear of the two closely spaced cross bars, and then clipping to the front cross bar.
- On height adjustable models, adjust the legs by sliding to the desired height and secure with the fixing e-clips.
- Set all legs to the same height.
- Check the seat is level and the product is stable before use



L99387
Fixed Height



L99399
Adjustable Height



L99405
Mobile Fixed Height



M00754
Mobile Adj. Height

5.0 Product Care

5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- These seats may be autoclaved on cleaning cycles up to 73°C
- Dry the seat thoroughly before use.