

### Maximum User Weight



### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.



F18864  
Adjustable Height



F19455  
Adjustable Height &  
Detachable Arms

### Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product. Any guarantee issued by NRS Healthcare does not affect your statutory rights. Before using the commode remove from packaging and examine for any visible damage. If you see any damage or suspect a fault, please contact your supplier for further instructions.

### Intended Purpose


Dovedale commodes are designed to assist those who have difficulty in accessing or using a standard toilet. It can also be used as a temporary chair. All Dovedale commodes have a hinged plastic toilet seat, commode bucket with lid and vinyl lift off seat.

### Instructions for Use:

Remove vinyl seat to use commode.

Replace vinyl seat to use commode as a temporary chair.

- 1 - To adjust the height, remove each 'E'clip in turn. Slide each leg to the required height.
- 2 - Align the holes and replace 'E' clip.
- 3 - Ensure that all legs are secured at the same position in order to prevent rocking.

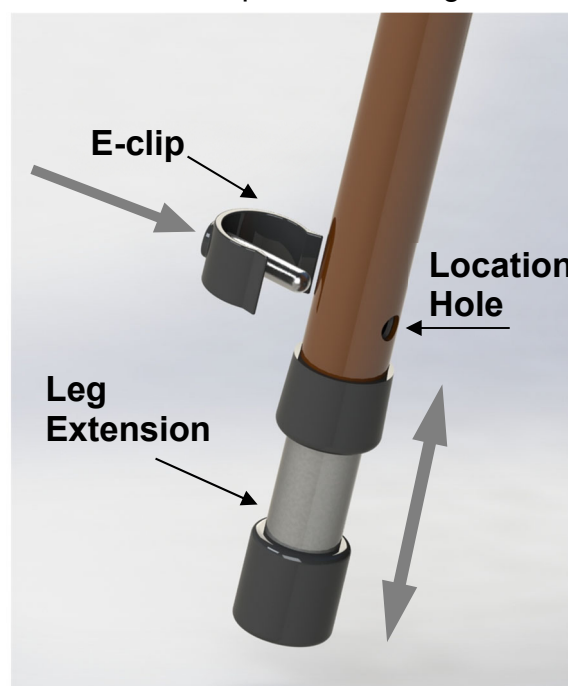

**Never extend legs in a manner so that the commode is slanting. This will compromise the safety of the user.**

### **Detachable Arms Model (F19455)**

1. To detach the arms for transfer. Using both hands press in the spring buttons on each end of the arm.
2. Remove the arm by lifting it from the two frame fixing sockets.
3. To refit the arm, locate into the frame fixing sockets, press in the spring buttons and push the arm down until the spring buttons "click" into position.

### **Commode Buckets**

To remove the commode bucket, simply lift the toilet seat and fix the lid provided onto the bucket to avoid splashing and carefully raise the bucket and dispose of the contents into a proper sluice or WC.

**Never use an open drain, sink or basin. This could cause a serious health risk.**

Clean out the residue with a suitable disinfectant and refit to the commode after use.

### **Customers Who Purchased This Product Also Purchased: L99351 Classic Bio Care Bags**

#### **NO Liquid, NO Odour**



*The new Bio-degradable Care Bag is a commode potty liner containing a special sheet that absorbs and gels waste. After using, the liner is simply removed, sealed with draw string and disposed, either by incineration or as the care bag is biodegradable into normal household rubbish. By gelling and containing the waste, the Care Bag reduces spills and contamination, eliminating the onerous and time consuming need to wash the commode potty. **Supplied in pack of 20.***

### **Cleaning**

To clean vinyl seats use a suitable detergent and warm water, wipe seats and dry with disposable paper towel. Clean steel frame and toilet seat with a suitable detergent and warm water, wash framework and arm rests thoroughly paying particular attention to the toilet seat hinges and the underside of the toilet seat. Can be autoclaved at 80°C. Allow equipment to dry prior to stacking and storage.

### **Please Note**

Under no circumstances should the commode be moved whilst occupied. It is advised that you check the frame of the product annually for signs of wear and tear, dimpled or flaking paint or cracks in the frame should be reported to your supplier immediately. Should there be any such evidence, please refrain from using the commode.

### **Local Authority and Health Service Employees**

These instructions must be left with the equipment user to refer to. If further copies are required, please contact Nottingham Rehab Supplies.

### **IMPORTANT**

For information on how to obtain the User Instructions in other languages and formats for use by people with visual, reading or cognitive impairments, please contact our customer services department.

Additional copies of this user instruction are available on our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)

### **NRS Healthcare**

Sherwood House, Cartwright Way  
Forest Business Park, Bardon Hill  
Coalville, Leicestershire. LE67 1UB UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservice@nrs-uk.co.uk](mailto:customerservice@nrs-uk.co.uk)

**NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:**

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**