

L96738

Wide Shower Stool

L96749

Extra Wide Shower Stool

### **General Safety Warning**

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### **Maximum User Weight**



## 1. INTENDED USE

The Wide and Extra Wide Shower stools are intended to provide a comfortable, stable, seat for larger users who are unable to stand during showering. The Wide shower stool is designed to be suitable for use with most standard shower trays.



L96738



L96749

## 2. SAFETY

**Warning:** The maximum user weight must not be more than 254Kg (40st).

Inspect product at regular intervals for signs of wear or damage.

Make sure the floor surface is level and even to avoid rocking.

Do not use if you suspect a fault.

## 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

## 4. IN USE

### 4.1 Stool Height Adjustment

Remove the plastic e-clips connecting the stool legs and leg extensions.

Slide the leg extensions to the required position.

**Note:** Make sure that all legs have been set to the same height.

Replace the plastic e-clips making sure that each e-clip is fully connected.

## 5. MAINTENANCE

Check that all fixing e-clips are inserted correctly.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately.

## 6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, neutral detergent and warm water.

Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

Dry thoroughly before use.

## 7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)